SUNBED

Assembly and operating instructions manual

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This user notice refers to Sunbed, the beach lounger for people with reduced mobility.

Read carefully the following instructions and always follow the safety rules.

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INSTALLATION

The PRM beach lounger can be used on hard surfaces (concrete, terraces) as well as soft surfaces (sand, grass)

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Step 1: Levelling the ground. On sand, a broom is all you need to level the ground and then place the deckchair where you want it to be.

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Step 2: Installation. Simply unfold the Sunbed and adjust the sunshade to your preference.

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WARNING: Despite the presence of a finger guard system, we recommend that you always handle the Sunbed with care when changing its position.

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UNINSTALLATION / DÉINSTALLATION

Step 1: Cleaning the Sunbed. One of the advantages of this deckchair is its structure, which prevents sand from accumulating and weighing the whole unit down. So sweeping up is easy.

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STEP 2: Uninstallation. All you have to do is fold the Sunbed and move it where you need it. Weighing just 11 kg, it is relatively easy to move around.

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WARRANTY / GARANTIE

DESCHAMPS offers a limited warranty of two (2) years from the date of purchase. This warranty only covers manufacturing defects in the product. Alterations due to normal use are not considered manufacturing defects. Damages caused by vandalism are not covered by the warranty. The guarantee does not apply to:

- normal wear and tear or improper use, maintenance or storage of the solution.
- accidents or natural disasters.
- repairs or modifications made to the solution without written consent of DESCHAMPS company.
- labour or shipping costs that may be necessary if repairs are contemplated.

If you are not satisfied with the terms and conditions or the repairs made, please contact Accessrec Europe directly. Products may not be returned without the consent of DESCHAMPS company. The two (2) years warranty is only applicable to the original purchaser.

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Property of ETS A.DESCHAMPS ET FILS and
DESCHAMPS MATS SYSTEMS INC. Mobi-mat®. All
rights reserved. Non-contractual pictures.
Products and specifications can be changed
without prior notice.

DESCHAMPS MATS SYSTEMS
218 Little Falls Road, Unit 12
Cedar Grove NJ 07009 – USA
+1 973 928 3040
GSA contract#47QSWA23D004U