


This user notice refers to Accessound, the headset to guide visually impaired people when swimming.
Read carefully the following instructions and always follow the safety rules.

CONDITIONS OF USE

Headset presentation and loading

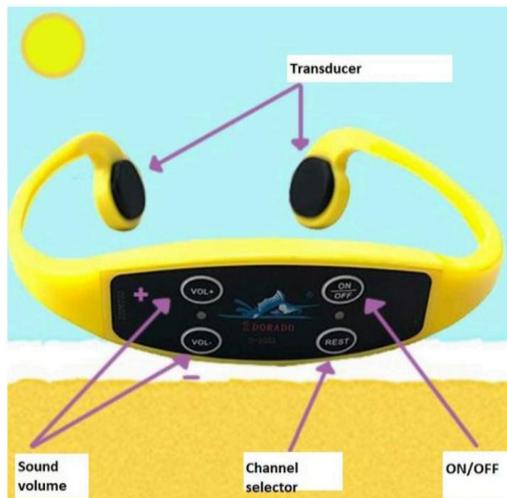

When you receive your Accessound, please make sure to plug in both units (headset and transmitter) and fully charge each unit before use:

- The headset uses DC + 5V
- The transmitter uses DC + 5V

First, it is necessary to ensure that the headsets are fully charged before each use: the charging time

- The red light indicates that the device is charging.
- The light off means the headset is fully charged and ready to use.

CAUTION: If the product is not used for a long period, the headset must be charged at least once a month to avoid any risk of damaging the battery and degrading its lifespan.

Starting the system

- Switch on the headphones by holding the on/off button pressed for 3 seconds.
- Turn on the transmitter (walkie-talkie) by turning the volume knob clockwise.

When the transmitter is on, 2 lights should appear:

- Blue light means the unit is on
- Red, blue or yellow light indicates which channel the transmitter is on
- Use the function button (REST) on the headphones to select the desired channel

The indicator lights on the headset and the transmitter must be the same color to ensure a good connection between the 2 devices. When 2 headphones are on the same channel, no extraneous sound should be heard.

- Raise or lower the volume according to your preference and comfort

The helmet must be installed in the opposite direction of a usual audio helmet, it is important to position the transducers at the level of the cheekbones and not the ears!

Tips for use and Safety

For proper use of the Accessound, we recommend that you use the caps available in your order to avoid any risk of loss in the event of waves. We advise you to always do a sound test with the user before entering the water to ensure that the latter is comfortable with the product.

Swimming is not done just anywhere, it is essential to respect the swimming areas supervised and secured by professional lifeguard. Do not hesitate to offer a flotation vest for more safety.

Important signs for communicating with a visually impaired person



WARRANTY

DESCHAMPS offers a two (2) year warranty from the date of purchase. This warranty covers manufacturing defects only. Alterations due to normal use are not considered manufacturing defects. Damage caused by vandalism or the is not covered by the warranty.

The warranty does not apply to:

- normal wear and tear or improper use, maintenance or storage of the solution.
- accidents or natural disasters.
- modifications made to the solution without the written consent of DESCHAMPS.
- repairs made without the written consent of DESCHAMPS.
- labor or shipping costs that may be necessary if repairs are contemplated.

If you are not satisfied with the conditions or repairs, please contact DESCHAMPS directly. Products may not be returned without the consent of DESCHAMPS. The two (2) year warranty is only applicable to the original purchaser.

REFERENCES



ETS A.DESCHAMPS ET FILS
Usine de Bourisson
16400 La Couronne – FRANCE
+33 (0) 545 677 030

Property of ETS A.DESCHAMPS ET FILS and
DESCHAMPS MATS SYSTEMS INC. Mobi-mat®. All
rights reserved. Non-contractual pictures.
Products and specifications can be changed
without prior notice.

DESCHAMPS MATS SYSTEMS
218 Little Falls Road, Unit 12
Cedar Grove NJ 07009 – USA
+1 973 928 3040
GSA contract#47QSWA23D004U