



Ingredients for maintenance



we do not endorse any special brand, just recommend to use grease.

Loosen the knob in order to show the thread.

Take the grease tube and squeeze a small amount of grease onto the thread.

Then tighten the knob. By tightening the knob, the grease will spread inside the housing and on the thread.

Sequence of this care: once a week



Loosen up the safety screw located on the housing of the floating arm chair

Take the oil bottle and insert few drops of oil.



Tighten the safety screw back



Sequence of this care: once a week

Loosen up the black knob

Remove the metal support by pulling upward



Squeeze the grease tube and poor some grease into the housing.



Put back the metal support and tighten the black knob



Sequence of this care: once a week

Pull down the chain that control the inclination of the back seat




Push the back seat forward to have a better access to the chain and its housing

Squeeze some oil into each end of the moving parts




Sequence of this care: once a week

Pull out the floating arm, by squeezing in the pin to free the arm

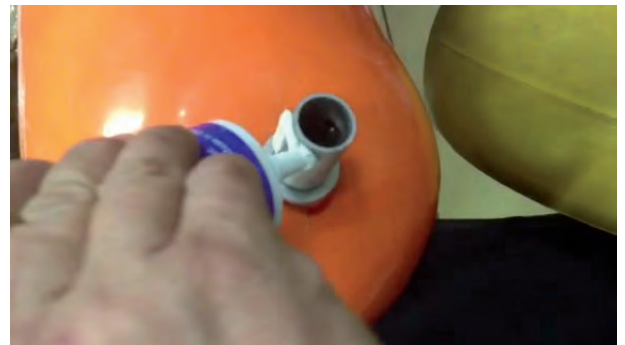

Pull out the rod and squeeze the tube and spread the grease around the entire rod

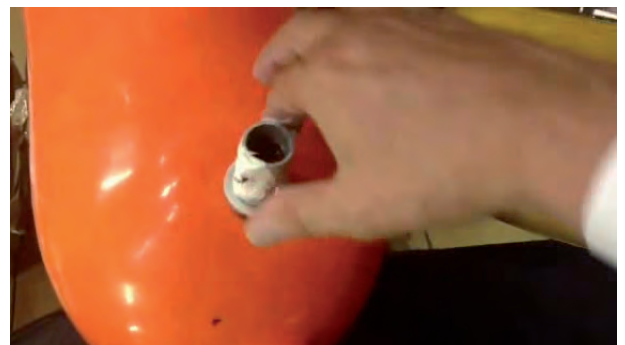





Re-insert the rod into the housing.



Sequence of this care: once a week


www.mobi-mat.com

 mobimat.access
 MobiMatUS
 recpath Mobi-Mat
 company/deschamps



DESCHAMPS
Usine de Bourisson
16400 La Couronne - France
☎ : + 33 (0) 545 677 030
✉ : info@mobi-mat.com

DISTRIBUTOR