



Although the Mobi-chair™ floats, it is not a boat. The sitting position is stable and you won't slip out of the Mobi-chair™ under normal conditions of use.

However, for safety reasons, a life jacket is required for non swimmers (this may also be required by law).  
Mobi-chair™ is not intended for areas with current or strong waves.



### ADVANTAGES



- > UV and sea water resistant
- > Mobile
- > Safe & robust
- > For both adults and children
- > Floating
- > Durable
- > Useful & ergonomic
- > Seamless transition from boardwalk-to beach-to water

## Technical specifications

Folding frame	Aluminum frame & stainless fork and hardware, salt water resistant			
Mobi-chair dimensions®	Width	Length	Height	Weight
	107 cm 42"	152 cm 60"	127 cm 50"	30kg 66 lbs
Seat dimensions	Width	Depth	Height	Max load capacity
	46 cm 18"	43 cm 17"	48 cm 19"	136 kg 300 lbs
Adjustable backrest	3 positions: relaxation, seated, swimming			
Seat color	Blue			
Seat Fabric	UV Resistant - Water Resistant Ventilated - Non Allergenic			
Detachable Armrest	PVC Flip up - detachable floating armrest - low pressure			
Front wheel	30cm x 16cm   12" x 6-1/2" Rubber Pneumatic Tire*1 w/quick release			
Rear wheel	40cm x 19cm   16" x 7-1/2" Rubber Pneumatic Tire*2 w/quick release			

## Adaptable & high quality wheelchair



- > **Easy mobility** through sand and other rough surfaces
- > **Safety harness**



- > **Rubber pneumatic** tires absorb the shock and discomfort caused by rugged terrain



- > **Maximum stability** with floating armrests and floating tires



- > **Handles integrated** into the armrests provide support
- > **Transportation and storage are convenient**

## Easy installation & low maintenance

### QUICK INSTALLATION


Assembly and disassembly of Mobi-chair™ without material and in 10 minutes maximum.

Completely foldable, Mobi-chair™ is easily transportable and fits perfectly in the trunk of a car.

### EASY MAINTENANCE

Once a week, lubricate all connecting rods and safety screws.

After each use, rinse Mobi-chair® with fresh water to avoid corrosion and store it in a clean, dry place.

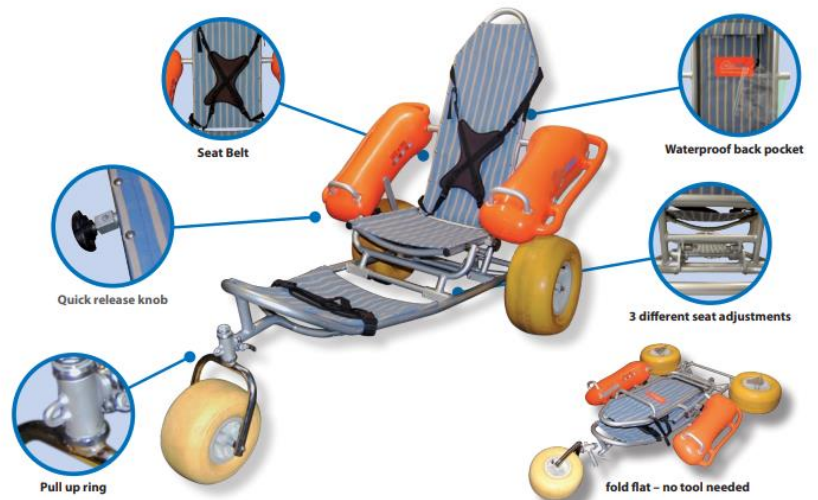

## Endless applications

SAND



OCEANS & SEAS



LAKES



SWIMMING POOLS



## What are our additional products ?

**Mobi-mat®**  
Accessibility mat



**Mobi-roller™**  
Mats dispenser



**Mobi-com™**  
Advertising mat



**Mobi-fence™**  
Sand fence



 [info@mobi-mat.com](mailto:info@mobi-mat.com)